





wcs-gmedtech.com

2 Tiers Stainless Steel Working Table WCS-T007

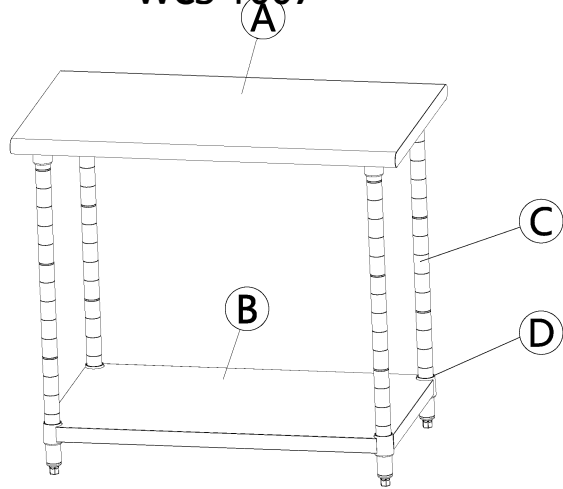


Assembly Steps:


- 1 Snap Plastic Clips (D) onto the pole grooves of each pole at the same position and make sure that the arrow of plastic clips are pointing up.

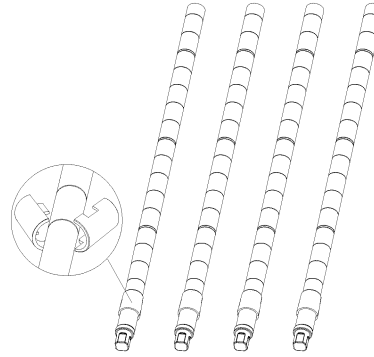


- 2 Hold the Stainless Steel Plate (B) on its side and slide it from the top of each pole down onto the plastic clips until there is snug fit.

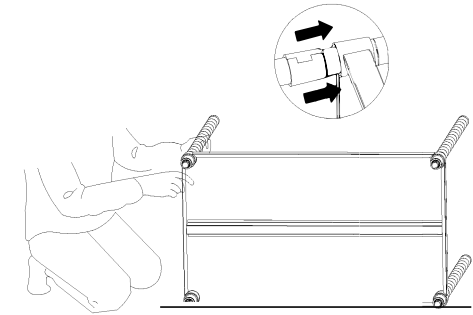

- 3 Set the rack upright. Repeat step 1. Snap plastic clips (D) into the top of each pole.

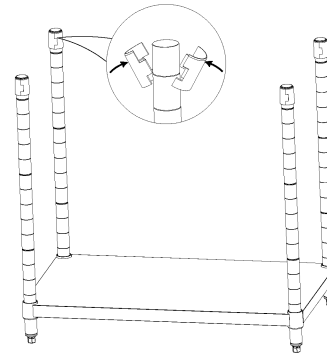

- 4 Put the Table Top (A) down onto the top of the rack until all plastic clips are well inserted the four collars welded under the table. Push firmly down on the shelf at its corner by rubber mallet to ensure the shelf in a place over plastic clips.

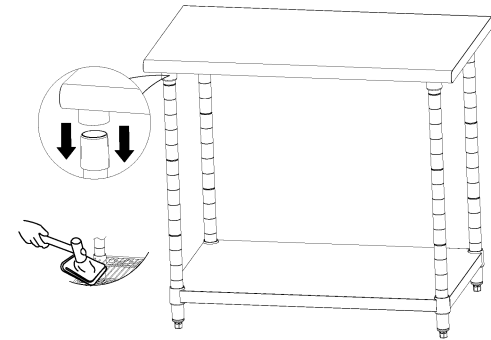

No.	A	B	C	D
Name	SS Table Top	Stainless Steel Plate	Pole	Plastic Clip
Parts				
Quantity	1PC	1PC	4PCS	16PCS